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1) Нажмите "1. Hard Lock Driver Installation".

🙀 Sentinel System Driver 5.41.1 (32-bit) - InstallShield Wizard		
License Agreement Please read the following license agreement carefully.		
Rainbow Technologies, Inc. License Agreement		
IMPORTANT: Read this before using your copy of Rainbow's software. This document is a legal agreement between you (either an individual or an entity) and Rainbow Technologies Inc Use of the software indicates your acceptance of these terms. As used in this License Agreement, the term "Software" means the Rainbow software included on the media or hardware provided with this License Agreement. 1. PROPRIETARY RIGHTS. The Software is a proprietary product of		
I accept the terms in the license agreement I do not accept the terms in the license agreement		
< <u>B</u> ack <u>N</u> ext > Cancel		

🙀 Sentin	el System Driver 5.41.1 (32-bit) - InstallShield Wizard 🛛 🔀
Destinati Click Nex	on Folder At to install to this folder, or click Change to install to a different folder.
	Install Sentinel System Driver 5.41.1 (32-bit) to: C:₩Program Files₩Rainbow Technologies₩Sentinel System Driver₩
InstallShield -	< <u>B</u> ack <u>N</u> ext > Cancel

Sentinel System Driver 5.41.1 (32-bit) - InstallShield Wizard	×
Setup Type Choose the setup type that best suits your needs.	
Please select a setup type. Complete All program features will be installed. (Requires the most disk space.)	
Custom Choose which program features you want installed and where they will be installed. Recommended for advanced users.	
istallShield	

🖟 Sentinel System Driver 5.41.1 (32-bit) - InstallShield Wizard 💦 🚺
Ready to Install the Program The wizard is ready to begin installation.
Click Install to begin the installation.
If you want to review or change any of your installation settings, click Back. Click Cancel to exit the wizard.
IMPORTANT:
Please remove all USB SuperPro keys before continuing!
InstallShieldCancel

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Video I	Device Driv	ver Install	ation	
	SKY T	V 2 Player V 2 Driver	Driver	
	CCL-2	15 / USB-/	225 Driver]
R Windo	ows Vistar CONTACT	USEXIT	SOMET	ECH

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U1.3M_ST - InstallShield Wiz	ard		X
	Welcome to the InstallShield The InstallShield Wizard will install	Wizard for U1.3M_ U1.3M_ST on your co	_ST mputer. To continue, click Next.
InstallShield	< <u>B</u> ack	Next>	Cancel

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U1.3M_ST - InstallShield W	izard
	InstallShield Wizard Complete
	The InstallShield Wizard has successfully installed U1.3M_ST. Before you can use the program, you must restart your computer.
	 Yes, I want to restart my computer now. No, I will restart my computer later. Remove any disks from their drives, and then click Finish to complete setup.
InstallShield	< Back Finish Cancel

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🖶 SkinSys - InstallShield Wizard	×
License Agreement Please read the following license agreement carefully.	4
End-user license agreement for SkinSys This SkinSys End-User License Agreement ("Agreement") is a legal agreement between you (either an individual or a single entity) and Sometech Inc,. ("Sometech") for the SkinSys software product identified above, which includes computer software and may include associated media, printed materials, and "online" or electronic documentation (collectively, "Software Product"). By installing, copying, or otherwise using the Software Product, you agree to be bound by the terms of this Agreement. If you do not agree to the terms of this Agreement, you may not use the	
I accept the terms in the license agreement I do not accept the terms in the license agreement	
InstallShield < <u>Back</u> <u>Next</u> > Cancel	

🕼 SkinSys - InstallShield Wizard	
Ready to Install the Program The wizard is ready to begin installation.	
Click Install to begin the installation.	
If you want to review or change any of your installation settings, click Back. Cl exit the wizard.	ick Cancel to
InstallShield	
< <u>Back</u> Install	Cancel

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ystem Properties ? 🗙		🚇 Device Manager		
System Restore General Compu Device Manager The Device M on your compu- properties of a Drivers Drivers Driver Signing compatible with how Windows Driver Signing Driver Signing Driver Signing Compatible with how Windows Driver Signing Driver	Automatic Updates uter Name Hardware lanager lists all the hardware devic uter. Use the Device Manager to o ny device. Device Manager to o iny device. Device Manager to o Device Manager to o Hardware to output to set up Hardware to output to set up Manager to output to output to output to set up Manager to output	Remote Advanced	File Action View Help Image: Sold State of the second st	
	OK Cance	Apply	Launches the Hardware Update	

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Hardware Update Wizard	
	Welcome to the Hardware Update Wizard
	This wizard helps you install software for:
	U1.3M_ST
2 Jun	If your hardware came with an installation CD or floppy disk, insert it now.
	What do you want the wizard to do?
	 Install the software automatically (Recommended)
ATTACK AND A DESCRIPTION OF A DESCRIPTIO	 Install from a list or specific location (Advanced)
	Click Next to continue.
	< Back Next > Cancel

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Ple	ase choose your search and installation options.
	O Search for the best driver in these locations.
	Use the check boxes below to limit or expand the default search, which includes local paths and removable media. The best driver found will be installed.
	Search removable media (floppy, CD-ROM)
	Include this location in the search:
	C:\Documents and Settings\admin\Desktop\dcs_pr 🚽 Browse
	Onn't search. I will choose the driver to install. Choose this option to select the device driver from a list. Windows does not guarantee to the driver you choose will be the best match for your hardware.

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Hardware Update Wizard	
Select the device driver you want to install for this hardware.	
Select the manufacturer and model of your hardware device and then click Next. If yo have a disk that contains the driver you want to install, click Have Disk.	u
Model Wodel WILLIAM_ST WILLIAM_ST WILLIAM_ST Wer01.11 WUSB Video Device	
This driver is digitally signed. <u>Tell me why driver signing is important</u> Have Disk.	
K Back Next > Canc	el

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System	Settings Change
?	Your hardware settings have changed. You must restart your computer for these changes to take effect. Do you want to restart your computer now?
	Yes No